

Nethealthbook.com

Systemic Mycoses

Dave Warshauer, PhD, D(ABMM)
Deputy Director, Communicable Diseases

david.warshauer@slh.wisc.edu

WISCONSIN STATE
LABORATORY OF HYGIENE

Systemic Mycoses

- *Histoplasma capsulatum*
- *Blastomyces dermatitidis*
- *Coccidioides immitis*
- *Coccidioides posadasii*
- *Paracoccidioides brasiliensis*
- *Penicillium marneffei*
- *Sporothrix schenckii*
- *Aspergillus species*
- *Emmonsia species*

WISCONSIN STATE
LABORATORY OF HYGIENE

Dimorphism

- **Majority systemic mycoses are caused by dimorphic fungi**
 - Molds in their normal soil environment, and on laboratory media when kept at 25-30°C.
 - Yeast when the temperature is raised to 37°C (as in the human host).
 - Nutritional factors such as certain amino acids can also enter into dimorphism, but are generally less important than temperature.

WISCONSIN STATE
LABORATORY OF HYGIENE

Histoplasma capsulatum var. capsulatum




Nethealthbook.com

WISCONSIN STATE
LABORATORY OF HYGIENE

Histoplasmosis

- Occurs throughout the world
- Endemic areas
 - Mississippi and Ohio River Valleys in the US.
 - Mexico, Central and South America
- Once thought to be a highly lethal form of pneumonia with up to 90% mortality
- Now known to be a rather common infection in endemic areas.

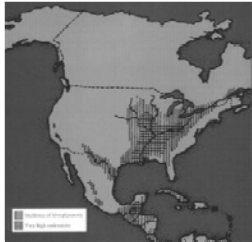

WISCONSIN STATE
LABORATORY OF HYGIENE

Histoplasma habitat

- Soil saprobe
- Loves droppings from bats or birds.
 - Requires high levels of creatinine and nitrogen
- Birds not susceptible to infection with Histoplasma
 - Likely related to their high body temperature
 - 40-42C (104-108F)
- Occupational risk for people working with chickens.
- Clearing Starling roosts has been associated with large outbreaks of fatal infections.

WISCONSIN STATE
LABORATORY OF HYGIENE

Histoplasma capsulatum Disease

- Usually self limiting flu-like illness and does not require medical intervention.
 - 90% asymptomatic
 - 4:1 male predominance for clinical disease
 - Infants and young children more likely symptomatic
 - Chronic pulmonary disease
 - Disseminated disease in immunocompromised

WISCONSIN STATE
LABORATORY OF HYGIENE

7

Pathobiology

- In human infection---small yeast 2-5 μ m in diameter.
 - predominantly in macrophages.
- Nonactivated macrophages do not effectively kill H. capsulatum and can actually spread the disease.
 - Can multiply intracellularly, kill the phagocyte, and infect additional cells

WISCONSIN STATE
LABORATORY OF HYGIENE

8


Pathobiology

- Granulomatous lesions in the lungs
 - Very similar to tuberculosis lesions
 - Lymphocytes, macrophages, Langhans' giant cells
- Severity of infection directly proportional to the number of conidiospores inhaled.
 - Miliary lesions when large numbers of the spores are inhaled.
- In most infections the cure is spontaneous and lasting immunity occurs.
- Histoplasma may remain viable and recurrence possible with decrease in CMI

WISCONSIN STATE
LABORATORY OF HYGIENE

9

Histoplasmosis

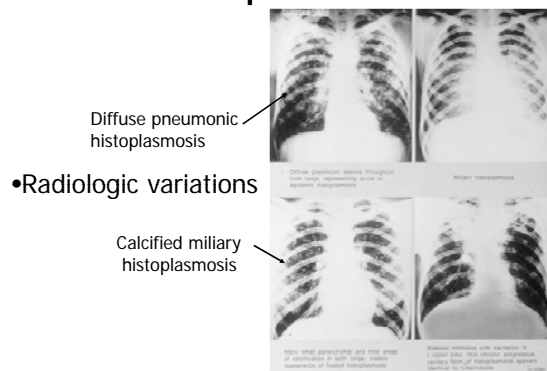


•Radiologic variations

WISCONSIN STATE
LABORATORY OF HYGIENE

10

Calcification

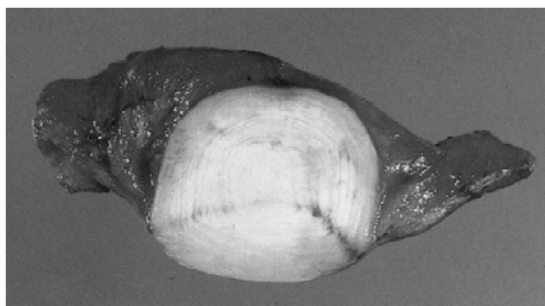WISCONSIN STATE
LABORATORY OF HYGIENE

11

Immunity and Treatment

- Immunity dependent on CMI.
 - Antibody is of little importance
- Healing of lesions leads to calcified granulomas similar to that seen in tuberculosis.
 - Old calcified nodules on chest x-ray not uncommon
- Treatment reserved for life-threatening infections
 - Amphotericin B
 - Itraconazole

WISCONSIN STATE
LABORATORY OF HYGIENE

12

Laboratory Diagnosis

- **Histology**
 - GMS stain
 - Wright stain of blood or bone marrow
- **Direct Microscopic Exam**
 - KOH, Calcofluor---2-5um yeast
- **Culture**
 - Enriched media (BHI with Blood, Yeast Extract Phosphate, Inhibitory Mold Agar)
 - 2-4 weeks at 30C
- **Antigen Detection**-----Urine EIA (miravistalabs.com)
- **Serology**
 - Complement fixation
 - EIA
 - Immunodiffusion
- **Real-time PCR**

WISCONSIN STATE
LABORATORY OF HYGIENE

13

Specimens for Fungal Infections

- Respiratory secretions, tissues, blood, CSF, other body fluids
- Discourage swabs
- Transport at room temp
 - Specimens with endogenous flora, refrigerate if >2hr delay
- Blood--Lysis centrifugation, BACTEC MYCO/F Lytic, or BacT ALERT MB
- CSF—large volume (10-20ml)
 - Centrifuge 2000g, 10 minutes
 - Inoculate pellet

WISCONSIN STATE
LABORATORY OF HYGIENE

14

Specimens for Fungal Infections

- Urine and other body fluids
 - Centrifuge 2000g, 10 minutes
 - Plate pellet
- Mince tissue, do not grind
 - Place 3-4 pieces on plate and press into agar
 - Exception for Histoplasma—want to grind to release intracellular organisms

WISCONSIN STATE
LABORATORY OF HYGIENE

15

Media for Primary Isolation of Systemic Fungi

- **Non-inhibitory media**
 - Sabouraud's dextrose agar
 - Potato Flake Agar
 - Potato Dextrose Agar
- **Selective Media**
 - Mycobiotic or Mycosel agar---cyclohexamide and chloramphenicol

WISCONSIN STATE
LABORATORY OF HYGIENE

16


Media for Primary Isolation of Systemic Fungi (2)

- **Enriched media w/ or w/o antibiotics**
 - Inhibitory mold agar---chloramphenicol and cyclohexamide
 - BHI with sheep blood w/wo antibiotics
 - Yeast extract phosphate agar with ammonia
- **Incubate plates or tubes at 30°C or 25°C**
 - Hold 4 weeks
- **For Blood Cultures**
 - Lysis Centrifugation
 - BACTEC MYCO/F or BacT ALERT MB






WISCONSIN STATE
LABORATORY OF HYGIENE

17


Silver Stain

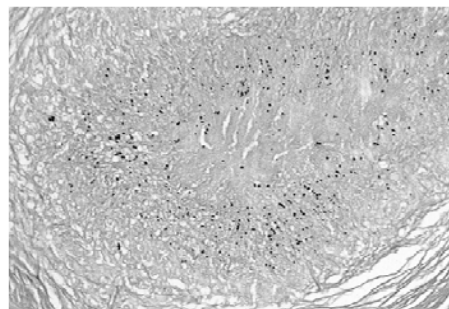WISCONSIN STATE
LABORATORY OF HYGIENE

18

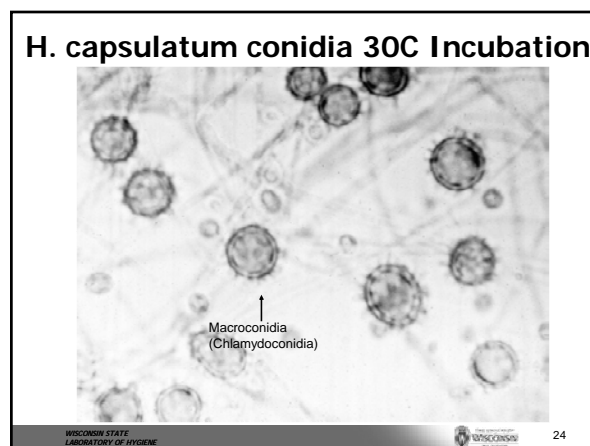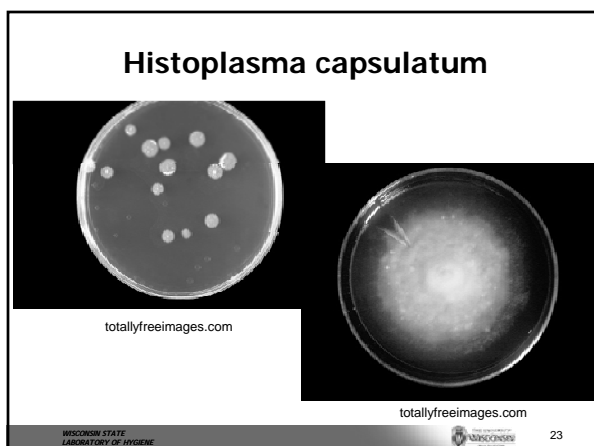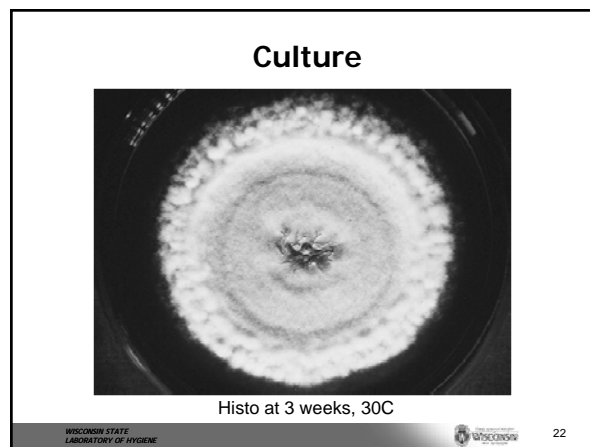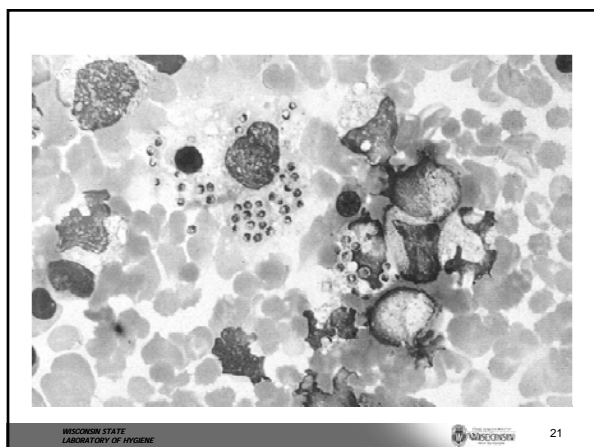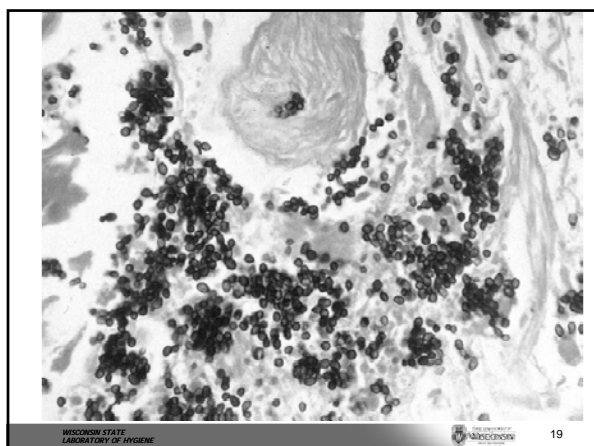

H. capsulatum

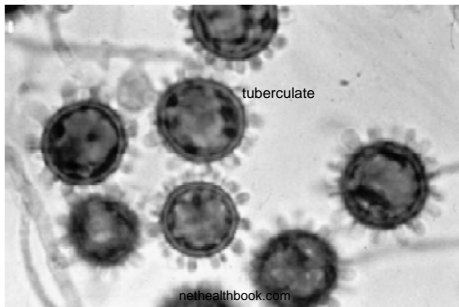


WISCONSIN STATE
LABORATORY OF HYGIENE



25

H. capsulatum—Yeast phase

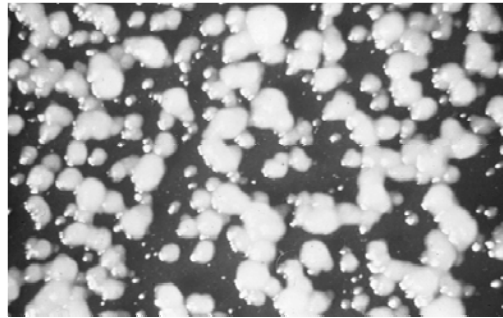

WISCONSIN STATE
LABORATORY OF HYGIENE



26

Differentiation from other Fungi

- Must differentiate from *Sepedonium* and *Chrysosporium* species that produce tuberculate macroconidia
 - More rapid growing
 - Not dimorphic
 - Usually will not grow in the presence of cycloheximide
 - Distinguish using DNA probe

WISCONSIN STATE
LABORATORY OF HYGIENE



27

Nucleic Acid Probe Identification

- GenProbe® Assay
 - Rapid
 - Chemiluminescent assay using labeled probes specific for each agent
 - Labeled DNA probe hybridizes with rRNA of the fungus
 - Available for *H. capsulatum*, *Blastomyces dermatitidis*, and *C. immitis*

WISCONSIN STATE
LABORATORY OF HYGIENE



28

BLASTO!



WISCONSIN STATE
LABORATORY OF HYGIENE



29


Blastomycosis

- *Blastomyces dermatitidis*
- Agent of North American Blastomycosis,
- Geographical distribution is similar to *H. capsulatum*
- More common in Wisconsin than *H. capsulatum*.

WISCONSIN STATE
LABORATORY OF HYGIENE



30

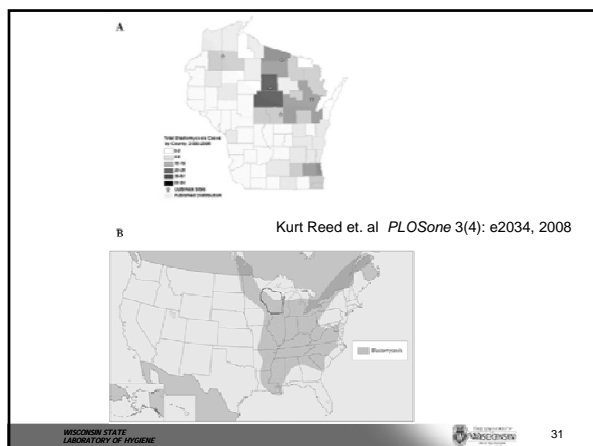

Blastomycosis

- The epidemiology is poorly understood
 - Lack of a good skin test reagent
 - Ecologic niche not well established
- Difficult to recover from the soil in endemic areas.
- Eagle River, Wisconsin outbreak 1985
 - First time *Blastomyces* isolated from the environment at the site of an outbreak
 - Isolated from soil containing decayed vegetative matter and from decomposed wood.

Clinical Manifestations

- **Two clinical presentations**
 - A **primary cutaneous infection** which usually remains localized to one area of the body
 - May indicate systemic disease
 - **Primary pulmonary infection** with possible secondary dissemination.
 - 30-45 day incubation
 - Mimics flu progressing to cough, weight loss, chest pain, low grade fever
 - 75% with isolated pulmonary disease
 - Infection may involve any organ
 - Secondary cutaneous infection
 - **Asymptomatic in >50% of those infected**

Systemic Disease

- **Common sites of infection in systemic disease**
 - Bones---long bones, ribs, vertebrae
 - Joints
 - Genitourinary tract---prostate, epididymis
 - CNS-----common in AIDS (40%), uncommon in immunocompetent (<5%)

Cutaneous Form

- A chronic suppurative granulomatous lesion.
- The presence of epithelial microabscesses and characteristic yeasts in the tissues is considered diagnostic.
- It is important to obtain urine and sputum samples from a patient with cutaneous blastomycosis since systemic spread may occur.




Histology


- **Body fluids or tissue specimens**
 - Look for the characteristic yeast form.
 - Large (8-15 μm) and thick walled.
 - The wall is prominent; "doubly refractile" on bright field microscopy.
 - A single daughter cell (bud) is present with a broad connection between the two cells (**BROAD-BASED BUDDING**).

WISCONSIN STATE
LABORATORY OF HYGIENE


37

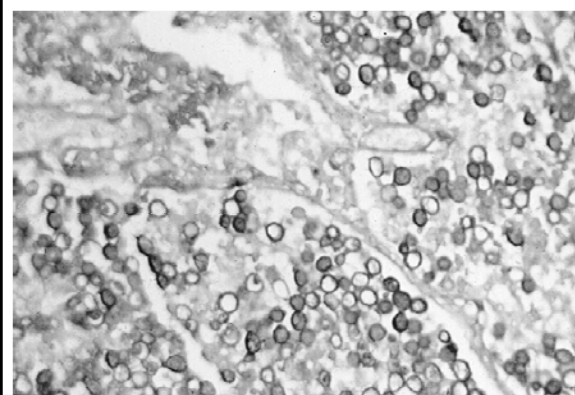WISCONSIN STATE
LABORATORY OF HYGIENE

38

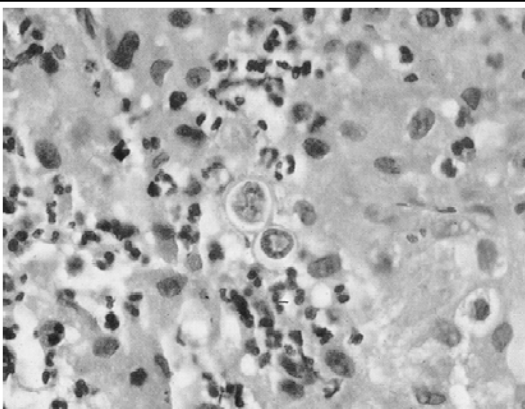
WISCONSIN STATE
LABORATORY OF HYGIENE

39

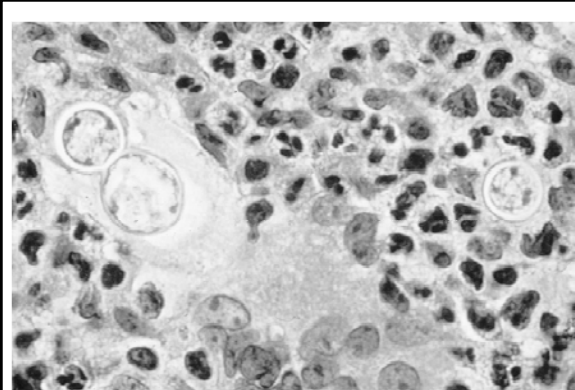
WISCONSIN STATE
LABORATORY OF HYGIENE

40

Direct Exam

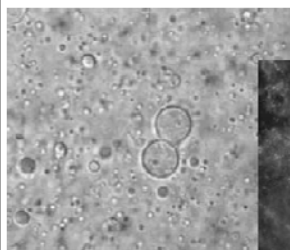

KOH Prep

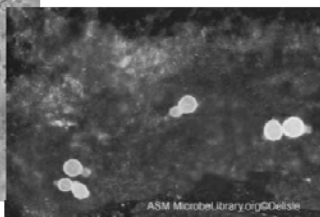


Calcofluor White

WISCONSIN STATE
LABORATORY OF HYGIENE


41

KOH Exam

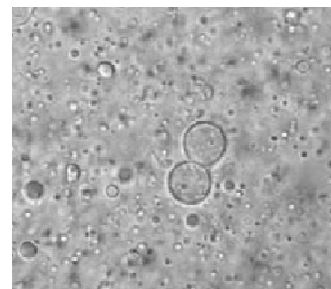WISCONSIN STATE
LABORATORY OF HYGIENE

42

Calcofluor White

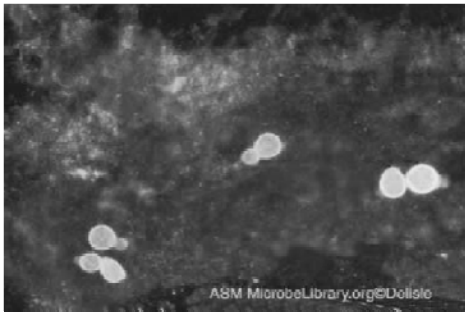


WISCONSIN STATE
LABORATORY OF HYGIENE



43

Gram Stain

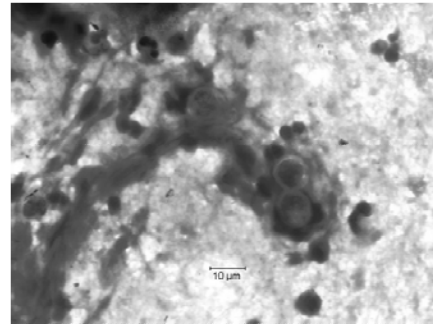

WISCONSIN STATE
LABORATORY OF HYGIENE



44

Culture Characteristics

- **On culture:**
 - Slow growing gray/white mold
 - Delicate, septate hyphae
 - Conidia usually absent on blood-containing media. May be sparse on PDA and SAB
 - "Lollypop" conidiation

WISCONSIN STATE
LABORATORY OF HYGIENE



45


Blastomyces dermatitidis



WISCONSIN STATE
LABORATORY OF HYGIENE



46

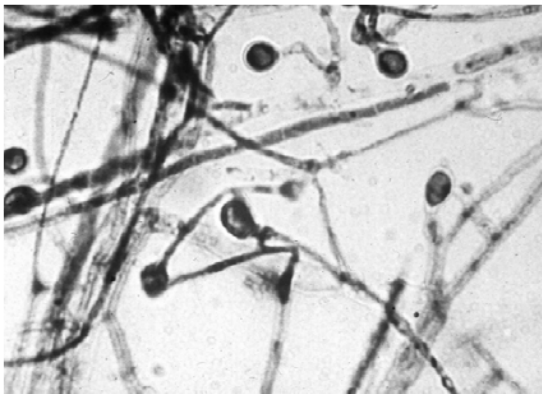

WISCONSIN STATE
LABORATORY OF HYGIENE



47

Blastomyces Mold Phase


- This form of conidia is also found in such fungi as *Chrysosporium* sp., *Pseudallescheria boydii* (*Scedosporium*), and various *Trichophyton* sp.
- Differentiation from these other species can be made by the following characteristics:
 - Slower growth
 - Growth in the presence of cycloheximide
 - Dimorphism
 - Nucleic acid probes

WISCONSIN STATE
LABORATORY OF HYGIENE



48

Blastomyces Yeast Phase

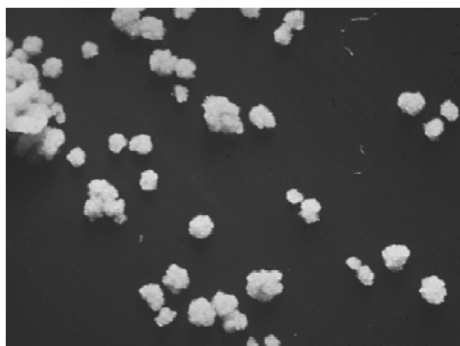


WISCONSIN STATE
LABORATORY OF HYGIENE



49

Blastomyces Conversion

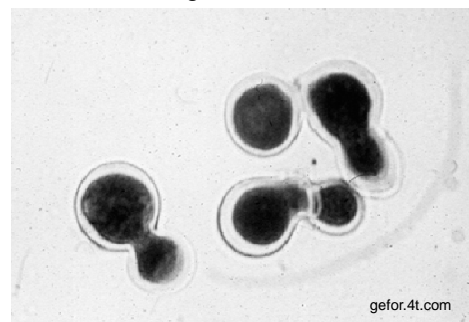

gefor.4t.com

WISCONSIN STATE
LABORATORY OF HYGIENE



50



botit.botany.wisc.edu

WISCONSIN STATE
LABORATORY OF HYGIENE



51

Coccidioides immitis and C. posadasii



Figure 18-2. Distribution and prevalence of coccidioidomycosis.




HEAD OF DOMINGO EZCURRA
FIRST CASE OF
COCCIDIOIDOMYCOSIS

WISCONSIN STATE
LABORATORY OF HYGIENE



52

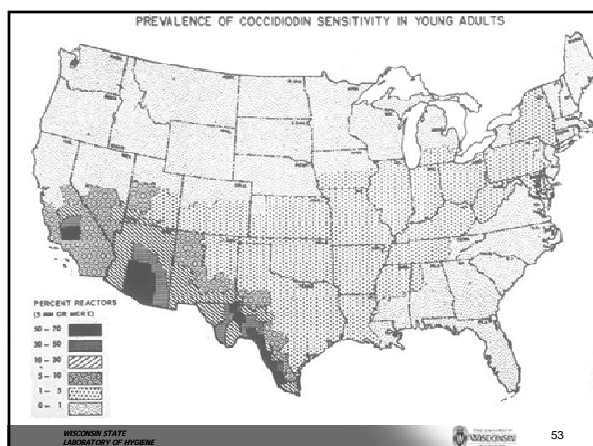

WISCONSIN STATE
LABORATORY OF HYGIENE



53

Coccidioides immitis

- Coccidiomycosis is sometimes known as "**San Joaquin Valley fever**". Up to 95% of the residents of the endemic area are skin test positive (coccidioidin test positive)
- Lower Sonoran Life Zone
 - Arid climate, hot summers, few winter freezes, low altitude, alkaline soil, sparse flora
 - Drought followed by heavy rains---Increased infections
 - 100,000 infected annually in U.S.
- Variety of animals infected
 - Positive cultures around rodent burrows
- Archaeology students discover new "infected" sites

WISCONSIN STATE
LABORATORY OF HYGIENE



54

Clinical Manifestations

- The primary disease is pulmonary, secondary to inhalation of small numbers of arthrospores
- Usually resolves spontaneously as an influenza-like infection.
 - 60% asymptomatic
 - 40% influenza-like illness, LRI or systemic illness
 - Cough, sputum, chest pain, malaise, fever, chills, night sweats, arthralgias, anorexia

WISCONSIN STATE
LABORATORY OF HYGIENE

55

Clinical Manifestations

- In a minority of cases a more chronic pulmonary infection occurs
 - Granulomatous lesions of the lung
 - Can lead to cavitation
- In rare cases (0.5%) dissemination occurs which can lead to rapidly fatal results.
- Reactivation infection occurs

WISCONSIN STATE
LABORATORY OF HYGIENE

56

Coccidiomycosis

Lesions variable:

Papules
Pustules
Plaques
Nodules
Ulcers
Abscesses

WISCONSIN STATE
LABORATORY OF HYGIENE


57

Histology

- Histological examination useful in confirming a diagnosis.
 - Spherules
 - 10-60 μm in diameter, but they may be as big as 200 μm .
- Immature spherules can be similar in size to the large yeast cells of *B. dermatitidis*
- The spherules contain **endospores** 2-5 μm in diameter
 - Similar in size to *Histoplasma capsulatum*
 - Will not see budding

WISCONSIN STATE
LABORATORY OF HYGIENE


58

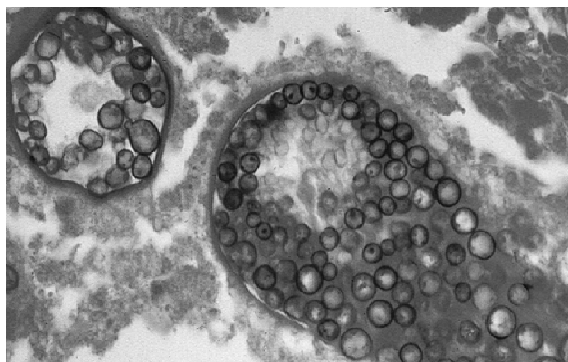WISCONSIN STATE
LABORATORY OF HYGIENE

59

Culture

- **Culture:**
 - The organism grows fairly rapidly.
 - Visible growth on Sabouraud's agar within a few days.

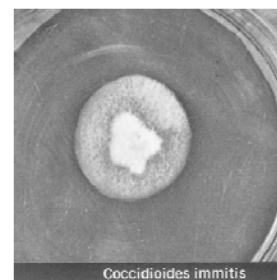

Coccidioides immitis

WISCONSIN STATE
LABORATORY OF HYGIENE

60


Microscopic Features

- **Arthrospores:**
 - Formed by fragmentation of hyphae
 - Very thick walled. Provides them with resistance to drying.
 - “Barrel-Shaped”
 - As a culture ages on media the entire hyphal mass may fragment and form arthrospores.
 - **Spores are extremely infectious - Handle with extreme care.**
- **Remember:** Arthrospores can be made by other fungi.
 - *Malbranchea* sp., *Gymnoascus uncinatus*, *Auxarthron* sp.
 - *Geotrichum* and *Trichosporon* can also form arthrospores
- Confirm the identification nucleic acid probe test

WISCONSIN STATE
LABORATORY OF HYGIENE

61

Coccidioides---Arthrospores

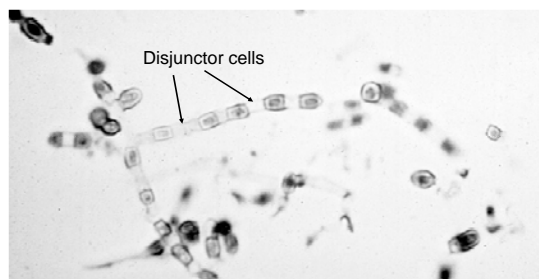WISCONSIN STATE
LABORATORY OF HYGIENE

62

Select Agent Regulations

- Report to CDC within 7 days of ID
 - Responsibility of lab confirming ID
 - Select Agent APHIS/CDC Form 4
- Secure against loss, theft, or release
- Destroy all subcultures and specimens
- Good News
 - Proposed to remove *Coccidioides* from SA list

WISCONSIN STATE
LABORATORY OF HYGIENE

63

Caution!

WORK SAFELY!

WISCONSIN STATE
LABORATORY OF HYGIENE

64

WISCONSIN STATE
LABORATORY OF HYGIENE

65