


# UW Cytogenetic Services

Wisconsin State Laboratory of Hygiene  
University of Wisconsin-Madison  
465 Henry Mall, Rm 419, Madison, WI 53706  
Phone: 608.262.0402 Fax: 608.265.7818

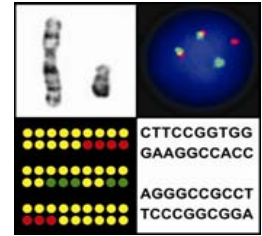

Thank you for your interest in the UW Cytogenetic and Molecular Genetics Laboratories.

Enclosed in this kit you will find all the materials necessary for sending a dried blood spot sample to our lab for testing.

1. Blood spot card
2. Biohazard bag
3. Pre-paid return mailer envelope

Please follow the provided instructions for collection. Failure to follow the provided instructions may result in the lab's inability to perform testing. Please contact your healthcare provider for assistance with this collection if needed.

The sample should be labeled with the patient's name, date of birth, and date of collection. Once the blood sample is collected, allow the card to air dry at room temperature for at least three hours. Seal the dried blood spot card in the biohazard bag and enclose the contents in the provided return mailer envelope.

The sample should be shipped to:

**UW Cytogenetic Services, 465 Henry Mall, Rm 419, Madison, WI 53706**

Please consult our website at [www.slh.wisc.edu/cytogenetics/](http://www.slh.wisc.edu/cytogenetics/) or call us directly at 608-262-0402 with any questions.